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Medical device
- a pure mineral gel

Intended use

Genital Herpes Gel is used for the external, physical and supportive treatment of herpes blisters.

Product properties

Genital Herpes Gel does not contain any greasy ingredients that favour bacteria and viruses. The formula is based on a mineral salt solution which breaks down into its natural components after application. The medical product supports and accelerates the healing of the blisters if it is applied in time at the first signs such as tingling and burning. Applying the genital herpes gel creates a protective film. The silicate contained in the gel absorbs exudates like a blotting paper due to its extraordinary binding capacity. It thus contributes to a reduction of the open and weeping areas. Due to its cooling effect, it can alleviate typical discomfort. The spread of infection-promoting germs is inhibited and healing is supported. Genital Herpes Gel is dermatologically rated as "very well" tolerated and clinically proven. Genital Herpes Gel is also suitable for allergy sufferers.

Instructions for use

Shake well before use! Clean hands thoroughly before and after use. Depending on your needs, Genital Herpes Gel can be applied thinly to the affected areas several times a day. Cotton swabs or the cleaned fingertip are suitable for this. For hygienic reasons, the tip of the tube should not come into contact with the blisters. Close the tube again after use. The product should be used until the herpes blisters are completely crusted or healed. There are no known undesirable side effects. A slight tingling sensation during application, which is often felt as pleasant, is a good indication of the physical effect of the product. Genital herpes gel is also suitable for allergy sufferers.

Contraindications / restrictions of use

For external use in cases of genital herpes. Genital Herpes Gel should not be used in cases of known allergies to any of the ingredients. Do not use at the same time as other care products such as creams, ointments and gels. For use during pregnancy, while breast-feeding and in people with severe immune deficiency, consult your doctor or pharmacist beforehand. If your symptoms worsen or do not improve after 5 days, you should consult a doctor in any case.

Safety information

Avoid eye contact. In case of eye contact, rinse with plenty of water. Use the product only until the stated expiry date and within 3 months after opening. Please report any potentially serious incident involving the product immediately to Promptmed (vigilanz@promptmed.de) and the competent authority in your country.

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Promotimed GmbH